


## **B** Product Specifications

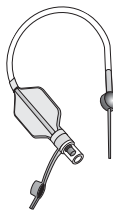

Model#	BE409
Type	Disposable Pilot Tube Repair Kit Assembly
Use	Single-patient use only, For endotracheal and tracheostomy tubes

### **CAUTION!**

Federal Law Restricts this Device to Sale by or on the Order of a Physician



### **Instrumentation**

Industries, Inc.

*We Make Respiratory Care Work!*

2990 Industrial Boulevard  
Bethel Park, PA 15102

**U.S. Toll Free:** 1-800-633-8577

**Business:** 412-854-1133

**Fax:** 412-854-5668

**E-mail:** [iiimed@iiirespiratory.com](mailto:iiimed@iiirespiratory.com)

**www.iiirespiratory.com**

ECO 1019 REV D



## **Instrumentation**

Industries, Inc.

*We Make Respiratory Care Work!*

### **Directions for Use**


### **Pilot Tube Repair Kit Assembly**

***Disposable - BE 409***




# A How To Use Pilot Tube Repair Kit Assembly


Figure

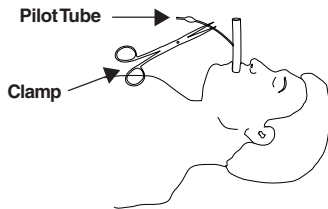

Figure

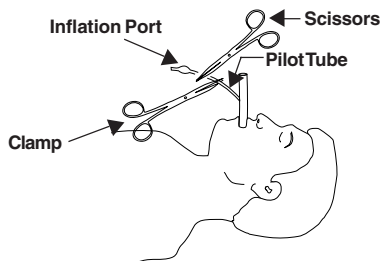

Figure

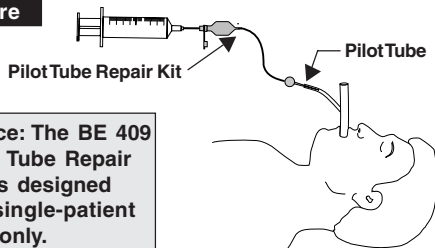

**Notice: The BE 409 Pilot Tube Repair Kit is designed for single-patient use only.**

This replacement assembly will quickly and effectively restore the integrity of a pilot tube line when the regular pilot tube is accidentally cut or when the inflation port is cracked and does not maintain cuff pressure. This assembly will help prevent the need for costly and potentially dangerous tube replacement. The pilot tube connector will fit the pilot line of most tubes. 0.032 in. O.D. (21g) to 0.050 in. O.D. (18g).

## To Replace or Repair A Pilot Line:

1. Clamp or bend pilot line proximal to the patient to maintain inflation of cuff. (*fig. 1*)
2. Cut off inflation port (*fig. 2*)
3. Insert Pilot Tube Repair Kit connector into cut pilot tube. (*fig. 3*)
4. Remove Clamp. (*fig. 3*)
5. Reinflate Cuff. (*fig. 3*)
6. Verify repair by monitoring cuff pressure.

## Helpful Hint:

It is recommended that the user confirm the proper fit of the **Pilot Tube Repair Kit Assembly** into a particular pilot tube line before a repair becomes necessary. This can be done most effectively by cutting the pilot tube of a used "trach" tube and connecting the **Pilot Tube Repair Kit Assembly**.