

Flowmeter Security Devices

Installation Directions

BE 167 — BE 168



**Instrumentation
Industries, Inc.**

2990 Industrial Blvd.
Bethel Park, PA 15102

US Toll Free: 1-800-633-8577

Business: 1-412-854-1133

US Toll Free Fax: 1-877-633-8661

Fax: 1-412-854-5668

E-mail: sales@iiimedical.com

www.iiimedical.com

10-13


ECO 2225 Rev. A

Installation Directions

1. Attach a **BE 167 Flowmeter Security Wire** or a **BE 168 Flowmeter Security Chain** to a **Flowmeter** by pushing the **Hook** onto the **3/8"** round portion of fitting. (see illustration)

NOTE: DO NOT attempt to attach the **Hook** directly behind the **Flow Control Knob** or onto a **HEX PORTION** of fitting.

2. Attach **Eyelet** end of device to wall or wall plate with screw.

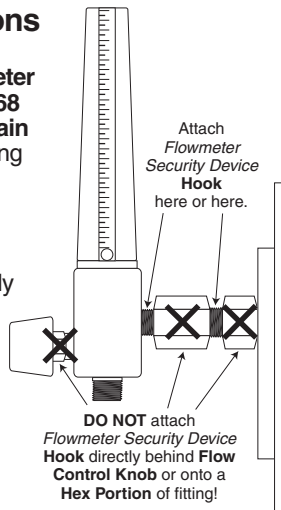

Made in **USA**