

D Product Specifications

Model #	SeCUREacord SJ 100 Series
Type:	Cord Cover
Use:	Single Patient Use
Materials:	— Polyolefin Plastomer — Nylon
Color:	Clear
Dimensions:	1" O.D x length
Packaged:	Sold Individually
Models & Length:	SJ 118 — 18 in. SJ 124 — 24 in. SJ 130 — 30 in. SJ 136 — 36 in. SJ 142 — 42 in. SJ 148 — 48 in.



Visit iiimedical.com/symbols.pdf
for the Glossary of Symbols used in
Instrumentation Industries, Inc. labeling.

Instrumentation Industries, Inc.

2990 Industrial Blvd.
Bethel Park, PA 15102
US Toll Free: 1-800-633-8577
Business: 1-412-854-1133
US Toll Free Fax: 1-877-633-8661
Fax: 1-412-854-5668
E-mail: sales@iiimedical.com
www.iiimedical.com




Instrumentation
Industries, Inc.

SeCUREacord Cord Cover

*Usage
Directions*

SJ 118
SJ 124
SJ 130
SJ 136
SJ 142
SJ 148

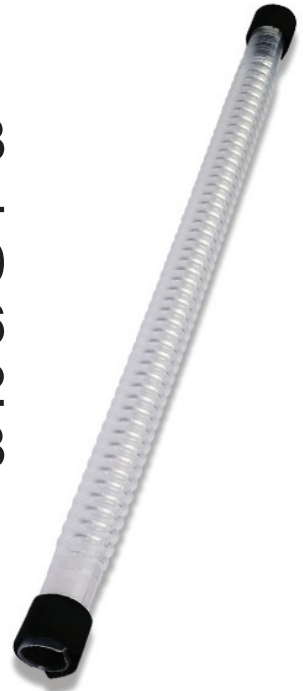

Single Patient Use

- Not made with
Natural Rubber Latex
- Not made with
Di(2-ethylhexyl)
phthalate (DEHP)



Patent Pending
12-20

ECO 2651 Rev. 0

A SJ 100 Series — SeCUREacord Description:


- The SeCUREacord cord covers are intended to gather and protect cords (heart electrodes, blood pressure monitor, pulse oxygen monitor, oxygen tubing, and intravenous line for fluids and medications), while limiting direct contact with a patient's skin and preventing the tangling of cords.
- All SJ 100 Series Cord Covers are for **Single Patient Use**.
- All SJ 100 Series Cord Covers are packaged as non-sterile.

B SJ 100 Series — SeCUREacord Notes:

- Do not reuse SJ 100 Series Cord Covers with a second patient.
- Ensure that security straps are not fastened so tightly that cords and tubing cannot slide through cord cover.
- Leave enough slack in cords and tubing to allow for normal patient movement.


C SJ 100 Series — SeCUREacord Installation & Usage

1. SeCUREacord shown closed (*figure 1*).

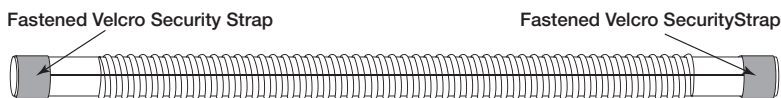


(figure 1)

2. Unfasten Velcro Security Straps on each end (*figure 2*).

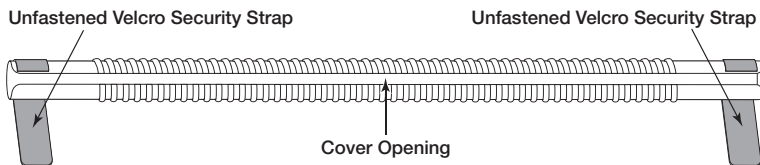

(figure 2)

3. Gather all cords and tubing and insert into SeCUREacord through opening in Cover (*figure 3*).

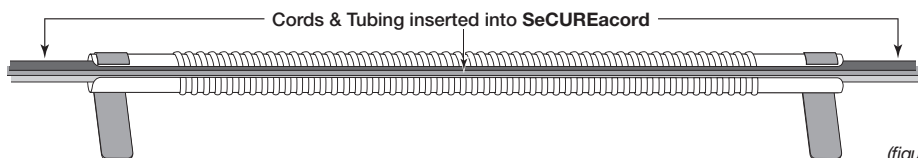

(figure 3)

4. After all cords and tubing has been inserted into cover, refasten Velcro Security Straps on each end (*figure 4*).



(figure 4)